VITA INTERNO

D

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GB

Instructions for use

Before using, please read the instructions for use.

VITA Interno includes colored, fluorescent ceramic materials which are used for internal coloring.

Indication:

The VITA Interno materials are suitable for all dental ceramic materials (e. g. VITA VM 7, VM 9, VM 11, VM 13, VM 15, VMK Master and titanium ceramic) and can be universally used.

Properties:

The VITA Interno materials are fine-grained and strongly fluorescent and are distinguished by their shade intensity. They can be mixed with one another, used either on their own or mixed into all dentine, enamel, translucent or effect materials.

Mixing:

The materials can either be mixed with Interno Fluid or mixed directly into the desired ceramic material (dentine, enamel, translucent or effect material). A max. proportion of 5 % of VITA Interno materials may be mixed into the ceramic materials in order not to alter the physical, chemical and, above all, the shade properties.

Application:

Depending on the application, the VITA Interno materials can be incorporated as mamelons in the incisal area, to achieve

effects in the neck or body area or to intensify the shade in various ceramic materials (see above).

Firing properties:

When using the VITA Interno materials, it is not necessary to select special firing programs.

It is only required to observe the parameters of the firing programs for the first or second dentine firing.

The firing parameters depend on the ceramic materials in use; see working instructions of the individual veneering ceramics.

More information on processing as well as an application video are available on our website at <https://www.vita-zahnfabrik.com>.

Storage:

Store the VITA Interno materials under dry conditions.

VITA Interno Fluid has a minimum shelf life of 72 months.

Supplementary information:

We cannot be held liable for damages resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of application. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product.

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VITA Zahnfabrik H. Rauter GmbH & Co.KG