VITA Bleachedguide 3D-MASTER®

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VITA Bleachedguide 3D-MASTER®

Dear Customer,
Congratulations on your purchase of your new VITA Bleachedguide 3D-MASTER!

VITA Bleachedguide 3D-MASTER is a new shade guide which assists you in the planning and monitoring of your patient's tooth whitening processes.

The VITA Bleachedguide 3D-MASTER follows the principles of the VITA SYSTEM 3D-MASTER based on the 3 shade parameters lightness, chroma and hue. In addition to the lightness and chroma levels, however, additional intermediate levels with supplementary shade samples have here been added (e.g. 0.5M1, 1.5M2 or 5M2.5) in order to obtain equidistantly spaced lightness levels that are clearly distinguishable by the human eye.

The lightness level is one of the key parameters with regard to color perception. It should be borne in mind that the chroma increases when the lightness decreases. The "M" on the shade samples stands for the middle hue, which is most suitable for the description of the color change.

The VITA Bleachedguide 3D-MASTER contains 15 shade samples, such as 1M2, whereby

"1" refers to **lightness level 1**
"M" refers to the middle (neutral) **hue**
"2" refers to **chroma level 2**

In order to make communication easier and more precise, the shade values of the VITA Bleachedguide 3D-MASTER were given a numerical designation from 1 to 29. This classification is based on the recommendations of the American Dental Association, which formed the basis for the new definition of the lightness levels.

The VITA Bleachedguide 3D-MASTER shade samples have been left unstained in order not to interfere with color perception.

Procedure

- Hold the VITA Bleachedguide 3D-MASTER at a distance of approx. 50 cm from the (open) mouth of the patient.
- Determine the current lightness level of the teeth, and express this according to either the VITA SYSTEM 3D-MASTER or the numerical values.

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GB

- Carry out the tooth bleaching procedure using the appropriate products.
- Check the result of treatment.
- In the case of discrepancies between the intended and the actual results, the treatment procedure can be repeated.

Tips for shade taking

- Perform shade taking, if possible, under daylight conditions or standardized daylight lamps and not under OP lamps.
- Make your selection swiftly, always accept the first decision since eye fatigue sets in after approx. 5 – 7 seconds.
- The entire surroundings should be kept in unobtrusive colors. That means no lipstick, no spectacles with tinted lenses, no brightly colored clothing unless this is concealed with a gray cape.

Hygiene tips and instructions for care

The acrylic parts of your VITA Bleachedguide 3D-MASTER are made of a high-quality, easy-care material that is temperature resistant (up to 180 °C). The entire Bleachedguide can be autoclaved at a temperature of up to max. 140 °C.

Disinfection

The VITA Bleachedguide 3D-MASTER can be surface disinfected. We recommend a customary, mild, spray disinfectant or disinfectant wipes. According to the manufacturers' product information these disinfectants are effective against HIV and HBV, and are available from most dental dealers. Please heed the respective manufacturer's instructions for use. Disinfectants containing phenol, compounds with phenylphenol groups or methyl ethyl ketone will damage the acrylic parts.

Note

The acrylic parts of the VITA Bleachedguide 3D-MASTER are neither monomer- nor UV-resistant, and must be protected from intense sunlight.

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