VITA ENAMIC® DISC

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VITA ENAMIC® DISC

The first hybrid ceramic DISC with a dual-network structure

Summary information

Date of issue: 07.15

Product description

- VITA ENAMIC is a hybrid ceramic with a dual-network structure. In this dental material, the dominant ceramic network is reinforced by a polymer network, with both networks fully integrated with one another.
- As a result, VITA ENAMIC is a hybrid dental material that combines the positive characteristics of both ceramics and composites. For the first time, this innovative hybrid ceramic guarantees particularly high load capacity after adhesive bonding in addition to a high degree of elasticity.
- The material is ideally suited to posterior crown restorations and also enables reduced wall thickness for minimally-invasive restorations.

Indication

Single-tooth restorations such as inlays, onlays, veneers and crowns, particularly for minimally-invasive restorations and restorations subject to high occlusal load (molar area).

Processing information for CNC machines

1. Mode of processing: milling

In order to process VITA ENAMIC by milling, the use of CVD tools is required. As a result of the considerable hardness of these CVD (chemical vapor deposited) diamond tools, the service life can be optimized in terms of cost.

In order to achieve a long service life, the correct parameters and strategies must be applied.

We recommend tools from:

Franken GmbH & CO.KG, www.franken-dental.com

- Tools for dental applications

VITA shade, VITA made.

VITA

En

Further information

- Detailed information on technical data, material layer thickness, preparation guidelines, etc. is provided in the VITA ENAMIC working instructions no. 1982.
- See instructions for use.
- Information on the CAD/CAM systems that are validated for VITA ENAMIC is provided under www.vita-enamic.com/www.vita-zahnfabrik.com/Systempartner

VITA Zahnfabrik has been certified in accordance with the Medical Device Directive and the following product bears the CE 0124 mark: **VITA ENAMIC®**

Rx only

VITA Zahnfabrik H. Rauter GmbH & Co.KG | Spitalgasse 3 | D-79704 Bad Säckingen | Germany
Tel. +49 (0) 7761/562-0 | Fax +49 (0) 7761/562-299 | Hotline: Tel. +49 (0) 7761/562-222 | Fax +49 (0) 7761/562-446
www.vita-zahnfabrik.com | info@vita-zahnfabrik.com | [facebook.com/vita.zahnfabrik](https://www.facebook.com/vita.zahnfabrik)

Recommended milling parameters:

Roughing

Tool	Ø 3 mm	Ø 2 mm
Speed [rpm]	20000–26000	32000–38000
Feed X Y [mm/min]	2000–2500	2000–2500
Feed Z [mm/min]	500	500
Infeed X Y [mm]	0.7–1.0	0.5–0.7
Infeed Z [mm]	0.2–0.4	0.2–0.3

Finishing

Tool	Ø 2 mm	Ø 1 mm
Speed [rpm]	32000–38000	40000–70000
Feed XY [mm/min]	1500–2000	1400–1800
Feed Z [mm/min]	500–1500	500–1500
Infeed XY [mm]	0.1–0.2	0.07–0.15

2. Mode of processing: grinding

In order to process VITA ENAMIC by grinding, ball and cone diamond grinding pins have proven helpful. The grain size used should be between D76 and D126.

We recommend tools from:

Franken GmbH & CO.KG, www.franken-dental.com

- Tools for dental applications

Recommended grinding parameters:


Tool	Ø 3 mm	Ø 2 mm	Ø 1 mm
Speed [rpm]	40000–50000	40000–60000	40000–70000
Feed XY [mm/min]	1200–1500	1200–1500	1000–1500
Infeed XY [mm]	0.1–0.15	0.1–0.15	0.08–0.1
Infeed Z [mm]	*	*	*

*No details on infeed depth (infeed Z).


Grinding tools should always be inserted in spiral (helix) fashion into the material; in doing so, the infeed should be constant (stepless).

Polishing

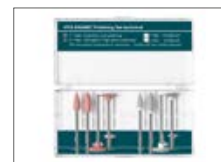In order to successfully polish VITA ENAMIC, special 2-stage polishing sets were developed. Perform high-gloss final polishing at the lowest speed and without cooling using water.

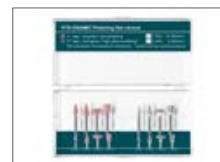Prod. no. EENPSETT/VITA ENAMIC Polishing Set technical for the handpiece

Prod. no. EENPSETC/VITA ENAMIC Polishing Set clinical for the contra-angle

Characterization of the shade

For shade characterization (staining) of VITA ENAMIC restorations, the specially developed VITA ENAMIC STAINS KIT is available with 6 shades and accessories. Please observe the information in the corresponding working instructions, (no. 1931).

Individualization of the shade

Shade individualization of VITA ENAMIC restorations can be performed using VITA VM LC flow or paste (layering technique), particularly in the transparent area of anterior restorations or in the vestibular area of posterior restorations. Please observe the working instructions for VITA VM LC (no. 1200) and for VITA ENAMIC (no. 1982).

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VITA VM LC flow

Note:

Restorations fabricated using VITA ENAMIC may not be fired under any circumstances during processing. Shade characterization and individualization are performed by way of polymerization.

VITA shade, VITA made.

VITA

920-26316 – 0715 (1.1) S – Version 001