VITA ADIVA® OXY-PREVENT

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Directions for Use

Mode d'emploi

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VITA ADIVA® OXY-PREVENT

DE

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VITA ADIVA® OXY-PREVENT

EN

VITA ADIVA OXY-PREVENT is a glycerine gel, permeable to light but impermeable to air, thus preventing oxygen inhibition when polymerising composites (the composite always comes into contact with the oxygen in the air during curing and does not cure completely on the surface). The product is used for covering exposed surfaces of adhesive composites as well as composite fillers prior to polymerisation. VITA ADIVA OXY-PREVENT enables the complete curing of composites right to the uppermost layer, and can be removed by spraying with water after use. VITA ADIVA OXY-PREVENT is also very suitable as a try-in paste.

Composition

Glycerine gel

Indications

For applying to the cementation gap of ceramic, hybrid ceramic or composite restorations before polymerisation of composite cement. For covering composite fillings before polymerisation. As try-in paste for more realistic assessment of the shade effect during the try-in of ceramic or composite restorations.

Contraindications

None known to date.

Application

1. For adhesion to ceramic, hybrid ceramic or composite restorations: Incorporate restorations with composite cement (e.g. VITA ADIVA F-CEM) and remove excess. Apply VITA ADIVA OXY-PREVENT to the restoration margins using application needle supplied. As VITA ADIVA OXY-PREVENT is permeable to light, the composite cement can be light-cured through VITA ADIVA OXY-PREVENT layer. Remove VITA ADIVA OXY-PREVENT with water spray and suction unit. Now finish and polish the restoration.

2. For composite fillings:

Apply VITA ADIVA OXY-PREVENT with an application needle to the modelled composite filling. The composite cement can be light-cured through VITA ADIVA OXY-PREVENT (see above). Remove VITA ADIVA OXY-PREVENT with water spray and suction unit. Now finish and polish the restoration.

3. As a try-in paste during the try in of ceramic and composite restorations:

Apply VITA ADIVA OXY-PREVENT to the cavity or the restoration with the application needle. Try-in, assessment of shade, checking the fit. Remove VITA ADIVA OXY-PREVENT with water spray. Now dry the restoration with an air-blower. Clean the restoration with phosphoric acid (e.g. VITA ADIVA TOOTH-ETCH), 37% solution of phosphoric acid. Rinse off the phosphoric acid thoroughly. Dry using an air blower.

Important!

When using low-viscosity adhesive composites, apply VITA ADIVA OXY-PREVENT carefully in order to avoid intermixing with, or displacing of the composite.

Storage

Store at room temperature between 10°C and 23°C.

Shelf life

The expiry date is 24 months for sealed (unused) syringes.

Explanation of symbols used on the packaging

 Symbol for "Batch designation"

 Symbol for "Use by"

 Symbol for "manufacturing date"

Presentation

VITA ADIVA OXY-PREVENT, syringe 3 ml, 10 disposable application needles

The product was developed for dental use and must be processed according to the instructions for use. The information/illustrations contained herein do not constitute an assurance of properties and are not binding. Date of issue of these instructions for use: 09.16
VITA Zahnfabrik is certified according to the Guideline for Medical Devices and VITA ADIVA OXY-PREVENT bears the designation 

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VITA Zahnfabrik H. Rauter GmbH & Co. KG