VITA VIONIC® WAX

DE

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VITA VIONIC® WAX

EN

VITA VIONIC® WAX is a millable blank made of a high-quality, millable and dimensionally stable wax with a high melting temperature.

VITA VIONIC WAX is available in the colors pink and white.

Composition

VITA VIONIC WAX consists of a mixture of natural and paraffin waxes.

Indication

VITA VIONIC WAX pink is suitable as a base material for try-in of digitally fabricated denture bases with acrylic teeth. It is possible to verify all parameters and to perform major adjustments if required. Then the wax denture can be completed using the packing or pressing technique and composite resin. VITA VIONIC WAX pink can be completely boiled out.

VITA VIONIC Wax white is used for the fabrication of trial dentures. They are entirely milled from wax (base including teeth) and are suitable to check the occlusion and phonetic and esthetic parameters.

Contraindication

VITA VIONIC WAX is not suitable for direct fabrication of definitive dental restorations.

Side effects

Side effects are not known to date.

Storage

At room temperature. Store in a dry location. Protect from direct sunlight and temperatures above 25°C.

Labeling

 Symbol for "date of manufacture"

 Symbol for "can be used until"

 Symbol for "batch number"

 Symbol for "storage temperature"

The batch number (LOT) is printed on the VITA VIONIC WAX disc.

Delivery form

VITA VIONIC® WAX pink and VITA VIONIC® WAX white are available in dimensions of Ø 98.4 / h 30 mm.

	color	size	Prod. No.
VITA VIONIC® WAX	white	Ø 98.4 / h 30 mm	ADW9830
	pink	Ø 98.4 / h 30 mm	ADP9830

Further information

Detailed information on processing VITA VIONIC WAX can be found in the VITA VIONIC working instructions.

 Please observe instructions for use!

The safety data sheet can be downloaded from www.vita-zahnfabrik.com or requested by fax at +49/0)77 61 /562-233.

VITA Zahnfabrik has been certified according to the Medical Device Directive and the following product bears the CE mark
CE VITA VIONIC® WAX

This product has been developed for dental applications and must be processed in accordance with the instructions for use. The information and illustrations provided do not constitute a guarantee of features and are not binding. Information issued: 12.2016.

VITA

VITA Zahnfabrik H.Rauter GmbH & Co.KG
Spitalgasse 3 · D-79713 Bad Säckingen
Germany · Allemagne · Alemania · Germania
Tel. +49(0)77 61 /562-0 · Fax +49(0)77 61 /562-299
Internet: <http://www.vita-zahnfabrik.com>
E-mail: info@vita-zahnfabrik.com
 facebook.com/vita.zahnfabrik