Gebrauchsinformation

VITA ADIVA® TOOTH-ETCH

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VITA ADIVA® TOOTH-ETCH

Phosphoric acid etching gel

VITA ADIVA TOOTH-ETCH is a 37% phosphoric acid etching gel for the total-etch technique VITA ADIVA TOOTH-ETCH displays excellent thixotropic behavior.

Composition

37% phosphoric acid on a thixotropic gel base.

Indications

Enamel etching or total-etch technique for:

- Adhesive bonding of inlays, onlays, veneers, crowns and bridges with a suitable luting system such as VITA ADIVA F-CEM and VITA ADIVA T-BOND.
- Composite filling
- Fissure sealings

Contraindications

The application of VITA ADIVA TOOTH-ETCH is contraindicated if the specified application technique is not possible.

Side effects

No known side effects

Use

1. Preparation of the syringe with VITA ADIVA TOOTH-ETCH

Screw off the sealing cap and replace with the application needle. **Please keep the cap!**

Note:

The application needle can also be bent into a slight arch to make application easier, if required.

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Precautionary measures for use

Squeeze out the first small drops onto a mixing block to ensure safe handling and to avoid the etching gel possibly shooting out due to potential trapped air.

2. Application of VITA ADIVA TOOTH-ETCH

Apply VITA ADIVA TOOTH-ETCH to the enamel and dentin surfaces, beginning with the enamel edges and allow to work for **20 s**. For milk teeth or highly fluoridated teeth, an etching time of **60 s** is recommended. Then rinse with water for **20 s** and dry with oil-free air. The dentin should **not** be dried too much as a slightly moist dentin surface is important for the function of the adhesive that is to be used subsequently (e.g. VITA ADIVA T-BOND). The etched tooth enamel edges must have a chalky white appearance.

Precautionary measures for etching

Do not contaminate the etched tooth enamel! If contamination does occur, it must be etched again as described above. Avoid contact of the etching gel with skin, mucous membranes and eyes. If accidental contact occurs, rinse with plenty of water.

Dispose of the single-use application needle after use and replace the sealing cap.

3. Application of the adhesive

Apply a suitable adhesive (e.g. VITA ADIVA T-BOND) with a disposable brush or microbrush onto the etched, saliva-free enamel and dentin surfaces according to the instructions for use.

Warnings

- VITA ADIVA TOOTH-ETCH contains phosphoric acid and is corrosive. Avoid contact with eyes, skin and mucous membranes (protective goggles are recommended for patient and dentist). In case of contact with eyes, wash immediately with plenty of water for at least 15 minutes and consult a doctor/ophthalmologist. In case of contact with skin, wash immediately with soap and water.
- Avoid contact with the gingiva, certain lining materials (based on glass ionomer) and the neighboring teeth.

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2. Application de VITA ADIVA TOOTH-ETCH

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