Gebrauchsinformation

VITA ADIVA® TE-CEM

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Instructions for use

VITA ADIVA® TE-CEM

Translucent temporary adhesive material

VITA ADIVA TE-CEM is an **esthetic, dual-curing, temporary adhesive material**. VITA ADIVA TE-CEM is translucent and shows good elastic properties. Due to a controlled slight expansion, the material seats well. Bonding to crown and bridge materials and to dentin is purely mechanical. VITA ADIVA TE-CEM can be easily removed from tooth stumps and crowns with the help of tweezers. Supplied in 4:1 double syringes, VITA ADIVA TE-CEM can be applied simply and comfortably. VITA ADIVA TE-CEM has no influence on the later curing of the adhesive composite used for permanent crowns and bridges or inlays or on acrylic-reinforced glass ionomer cements.

Composition

Prepared from multifunctional methacrylates, zinc oxide and organic acids

Indication

VITA ADIVA TE-CEM is suitable for bonding of temporary crowns and bridges and for temporary bonding of permanent restorations.

Use

Preparation of the VITA ADIVA TE-CEM syringe

Remove the cap on the double syringe (**dispose of it, do not re-use it**) and replace it with a 4:1 mixing tip. Secure the mixing tip by turning it 90° to the side. The material can be applied directly from the mixing tip.

Comment:

The used mixing tip can be used as a cap until the syringe is needed again. The first 2-3 mm of material that come out of the mixing tip should be thrown away. This is valid for each new mixture.

Instructions for use

Remove the cap on the double syringe (**dispose of it, do not re-use it**) and replace it with a 4:1 mixing tip. Secure the mixing tip by turning it 90° to the side. The material can be applied directly from the mixing tip.

Comment:

The used mixing tip can be used as a cap until the syringe is needed again. The first 2-3 mm of material that come out of the mixing tip should be thrown away. This is valid for each new mixture.

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Application

Apply under light pressure directly from the mixing tip onto the temporary crown or bridge. Apply a very small amount of mixed material, sufficient for a thin layer, onto the bonding surface of the crown or bridge that is to be temporarily bonded, and position it carefully in situ after no longer than **2:00 min** (processing time at 23°C (73°F)) first with light pressure and then with firm pressure.

Important information:

Do not dry the stump area beforehand. A slightly moist surface makes it easier to remove the temporary crown or bridge later. Alternatively the stump can be isolated with a thin layer of Vaseline or glycerine gel.

If the stump has been constructed with a stump construction material, the inhibition layer that may be present on the surface should be removed with a suitable solvent (e.g. ethanol).

In order to quickly fix the restoration, VITA ADIVA TE-CEM can be cured at the edge areas for **20 s** per surface. With translucent materials, curing can be done through the material (**40 s** per surface). Without light VITA ADIVA TE-CEM cures in the mouth within approx. **4 min**.

Remove any excess after curing. Remove a possible lubrication layer with ethanol.

Warnings

- Contact with skin should be avoided, especially for people who have a known allergic reaction to acrylate.
- Flush with copious amounts of water in case of contact with skin or eyes.
- If swallowed, consult a physician.

Storage information

Do not store above 25°C (77°F). Do not use after the expiration date.

Warnings

- Contact with skin should be avoided, especially for people who have a known allergic reaction to acrylate.

- Flush with copious amounts of water in case of contact with skin or eyes.
- If swallowed, consult a physician.

Storage information

Do not store above 25°C (77°F). Do not use after the expiration date.

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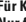
VITA ADIVA TE-CEM, 5 ml double syringe, translucent shade.

Guarantee

Harvard Dental International GmbH guarantees that this product is free of material and manufacturing defects. Harvard Dental assumes no further liability, nor any implicit guarantee concerning market-ability or suitability for a particular purpose. The user is responsible for the application and the proper use of the product. If defects occur on the product within the warranty period, the customer's only claim and the only obligation of Harvard Dental is the repair or replacement of the Harvard Dental International product.

Limitation of liability

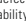
Insofar as exclusion of liability is legally allowed, Harvard Dental International GmbH bears no liability for losses or damage from this product, regardless of whether direct, indirect, special, concomitant, or consequent damage is concerned, regardless of the legal basis, including guarantee, contract, negligence, or premeditation.

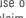
Always store out of reach of children!

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