Kurzinformation

Stand: 06.17

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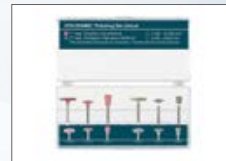
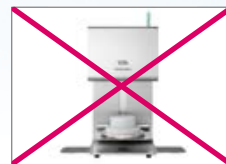
System and software requirements

VITA ENAMIC:

- **CEREC /inLab MC XL, MC X5 (Sirona)**
A **VITA Material Service Pack CEREC SW 4.0.x** can be downloaded for **CEREC software versions > 4.0 and < 4.2** from the Sirona website at www.sirona.com/de/CERECdownloads. For **inLab software versions > 4.0 and < 4.2**, a **VITA Material Service Pack inLab SW 4.0.x** can be downloaded from the Sirona website at www.sirona.com/de/inLabdownloads.

To mill VITA ENAMIC also in the small CEREC or inLab milling systems and with previous software versions (< 4.0), please select VITABLOCS Mark II, I14 or I10.

VITA ENAMIC *multiColor*:

Hardware requirements:

CEREC MC, MC X and MC XL Premium Package.
inLab MC XL. Processing with the inLab MC X5 machine is currently not possible.

Software requirements:

CEREC 4.5 or higher, CEREC Premium (CAM) 4.5 or higher, inLab (CAM) 16.1 or higher.

- **Milling additive - Dentatec** (relating to a full tank)

1. CEREC and inLab / 50 ml
2. CEREC MC, MC X, MC XL and inLab MC XL / 75 ml
3. inLab MC X5/250 ml

- **MyCrown Mill (FONA Dental s.r.o.)**

Can be processed with MyCrown Design 1.0 or later.

- **Ceramill Motion II (Amann Girschbach AG)**

VITA ENAMIC is processed with Amann Girschbach's Ceramill Motion 2 system using software release Ceramill Mind version 2.1.9 (XP) version 2.3.4 (Win7) and Ceramill Match 2 version 2.1.5 (Win7) or later.

VITA – perfect match.

VITA

- **KaVo ARCTICA/Everest (KaVo Dental GmbH)**
VITA ENAMIC is available in KaVo multiCAD, software version 2.8.0 or higher and can be processed using the KaVo ARCTICA Engine, version 2.5.9 or higher and using the KaVo Everest Engine, version 9.3 or higher.
More information is available at: www.kavo-everest.com
- **IOS Technologies (Glidewell Laboratories)**
VITA ENAMIC can be processed in the TS150 milling system using the latest version.
- **PlanMill (D4D Technologies LLC)**
VITA ENAMIC can be processed in the PlanMill 40 system using the latest software version.

UNIVERSAL SOLUTIONS*

VITA ENAMIC blocks with universal holder system are available for the following CAD/CAM systems:

- CORITEC Serie (imes-icore GmbH)
- CS 3000 (Carestream Inc.)
- DMG ULTRASONIC Serie (DMG Mori AG)
- Impression K-/S-Modelle (vhf manufacture AG)
- MILLING UNIT M Serie (Zirkonzahn S.r.l.)
- Organical Desktop Serie (R+K CAD/CAM Technologie GmbH & Co. KG)
- Roland DWX Serie (Roland DG)
- Röders RXD Serie (Röders GmbH)
- Zfx Inhouse5x (Zfx GmbH)
- ZENOTEC select (Wieland Dental + Technik GmbH & Co. KG)

*The CAD/CAM system partner has been validated by VITA Zahnfabrik for processing VITA ENAMIC.

For more information, visit www.vita-enamic.de

Please observe instructions for use

VITA Zahnfabrik has been certified in accordance with the Medical Device Directive and the following product bears the mark 0124 : VITA ENAMIC®

Rx only

Characterization of the shade

For shade characterization (staining) of VITA ENAMIC restorations, the specially developed VITA ENAMIC STAINS KIT is available with 6 shades and accessories.

Please observe the corresponding working instructions.

Individualization

The materials of the VITA VM LC CREATIVE Kit flow Edition, Prod. No. CVLFCCK are available for individualizing (layering technique) restorations made from VITA ENAMIC.

Please observe the corresponding working instructions.

Polishing

In order to successfully polish VITA ENAMIC, special 2-stage polishing sets were developed. Perform high-gloss final polishing at the lowest speed and without cooling using water.

- ⚠ **Note:** Restorations made from VITA ENAMIC should not be fired during processing under any circumstances. Shade characterization and individualization are performed by way of polymerization.

VITA Zahnfabrik H. Rauter GmbH & Co. KG
Spitalgasse 3 | D-79704 Bad Säckingen | Germany
Tel. +49 (0) 7761/ 562-0 | Fax +49 (0) 7761/ 562-299
Technical Hotline: Tel. +49 (0) 7761/ 562-222 | Fax +49 (0) 7761/562-446
www.vita-zahnfabrik.com | info@vita-zahnfabrik.com

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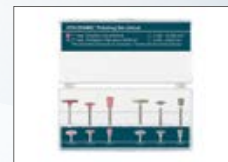
Prod. No. CVLFCCK /
VITA VM LC CREATIVE KIT flow Edition

Prod. No. EENPSETCV1/VITA ENAMIC
Polishing Set clinical/for the contra-angle

Prod. No. EENPSETT/VITA ENAMIC
Polishing Set technical/for the handpiece

VITA – perfect match.