

VITA AKZENT® Plus

Quickstart Working Instructions – PASTE/SPRAY



CTE-independent ceramic stains
for finishing, coloring and glazing.

VITA – perfect match.

VITA

VITA AKZENT® PLUS – FACTS & FEATURES

VITA AKZENT Plus stains and glaze materials are:

- CTE-independent ceramic stains for all standard veneering ceramics, monolithic zirconia, silicate or feldspar ceramics.
- for finishing, coloring and glazing
- for sealing the ceramic surface

When using VITA Akzent Plus with CHROMA STAINS you will benefit from:


- simple adaptation of chroma in the corresponding shade groups
- superior processing characteristics, such as simple application
- highly esthetic results through surface staining

SYSTEM COMPONENTS



VITA AKZENT® Plus – GLAZE SPRAY

📌 Use of GLAZE/GLAZE LT SPRAY

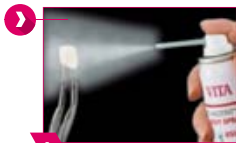

✔ Application of GLAZE:

- Clean the restoration
- Shake spray can well, immediately before use
- A distance of approx. 10 cm between the spray head and the surface to be sprayed is required
- Spray from all sides to apply a uniform layer
- Place the restoration on a firing tray for (stains firing and) glaze firing and fire according to the firing parameters
- In cases of insufficient brilliance, the process can be repeated

❗ **Please note:** do not apply glaze to inner surfaces!



➤ Optimal layer thickness



➤ Not enough
GLAZE applied



➤ Too much
GLAZE applied

VITA AKZENT® Plus – GLAZE PASTE

Use of GLAZE/GLAZE LT PASTE



Application of GLAZE:

- Use spatula to mix stains paste thoroughly
- Use enclosed brush to take up the paste directly from the small jar
- Spread the glaze material across the entire surface of the restoration to achieve uniform coating
- Place the crown to be glazed on the firing tray
- Place the restoration on the firing tray in the center of the furnace and use suitable firing program (depending on the underlying material) for firing

Please note:

- VITA AKZENT Plus PASTE FLUID is mixed with dried pastes to obtain the original consistency.
- Mix the liquids completely with the paste and make sure to avoid the formation of puddles.

VITA AKZENT® Plus – Stains

Use of stains and glaze




Application of GLAZE:

for stains fixation firing:

- Use spatula to mix the stains thoroughly
- Add color accents to the desired areas of the restoration
- Place restoration on the firing tray and fire
- Then apply GLAZE PASTE according to the instructions and fire again

for stains and glaze firing:

- Apply GLAZE PASTE according to the instructions
- Apply EFFECT, BODY or CHROMA to the glaze layer
- Place restoration on the firing tray and fire

 **Please note** : After stains fixation firing, the stains appear dull. To see the effect when applying the stains, wet the surface with a small quantity of liquid.

VITA AKZENT® Plus – Product portfolio

VITA AKZENT Plus EFFECT STAINS

- highly chromatic and opaque stains
- to reproduce individual shades and to adapt the shade to the characteristics of a natural shade
- when applied thickly, they will cover the underlying base shade completely

VITA AKZENT Plus BODY STAINS

- translucent finishing materials that have the effect of a thin color filter on the base material
- color changes and color shifts towards reddish, yellowish, orange, brownish or grey-greenish

VITA AKZENT Plus CHROMA STAINS

- glazing stains for modifying the chromaticity of the base material within a shade group

GLAZE

- Glaze material for a brilliant, glass-like, homogeneous and dense surface after firing.


GLAZE LT

- LT = Low Temperature: low-melting glaze material for ceramics with firing temperatures below 800 °C

Finishing Agent

- for a natural surface texture with a matte finish

VITA AKZENT® Plus – Available stains

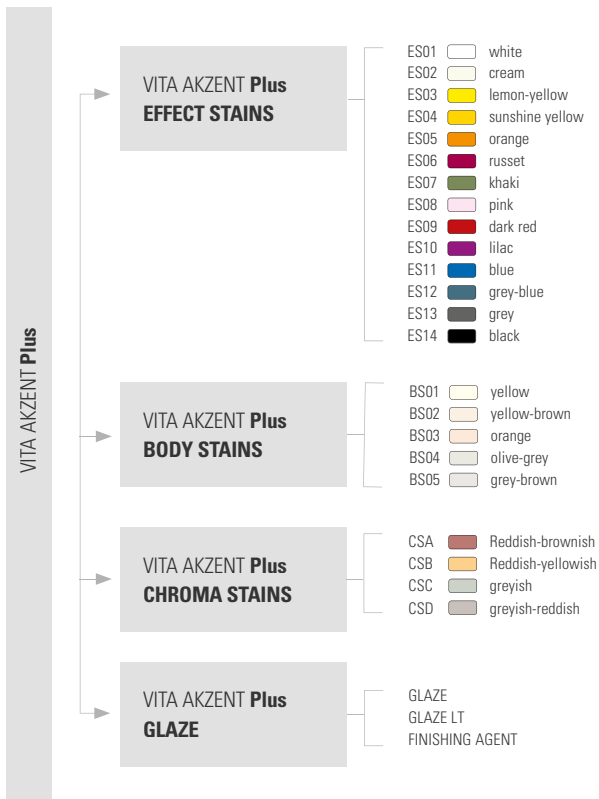



TECHNICAL HOTLINE

Questions about VITA product solutions?

Experienced dental technicians and technical consultants

will be glad to answer your questions:

- ▶ **Phone +49 (0) 7761/56 22 22**
- ▶ **Mail info@vita-zahnfabrik.com**
- ▶ **8 a.m. to 5 p.m. CET**



ONLINE SUPPORT

Questions about processing and product solutions?

You can find competent answers online in the form of **FAQs, videos and electronic tutorials.**

More detailed information can also be found in the comprehensive VITA AKZENT Plus Working Instructions No 1925.

- ▶ **www.vita-zahnfabrik.com**
- ▶ **www.facebook.com/vita.zahnfabrik**



Video tutorials can be found in the control unit of VITA SMARTFIRE.



VITA Zahnfabrik H. Rauter GmbH & Co.KG
Spitalgasse 3 · D-79713 Bad Säckingen · Germany
Tel. +49(0)7761/562-0 · Fax +49(0)7761/562-299
Hotline: Tel. +49(0)7761/562-222 · Fax +49(0)7761/562-446
www.vita-zahnfabrik.com · info@vita-zahnfabrik.com