VITA Sicherheitshinweise

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VITA Safety Information

Note for startup

- The operation of the devices is only permitted if the respective operating manual/safety information has been completely read and understood and the procedures described have been observed. Any other or additional use, e.g., processing of products other than those intended is considered misuse, as is the handling of hazardous materials or substances that are dangerous to personal health.
- Avoid electrical multiway socket outlets with extensions, as there is a risk of fire with overload.
- Never operate the firing device without the firing base installed.
- When the device is connected, do not reach into the open firing chamber, as there is a risk of electric shock or burns from touching live or hot parts.
- Do not operate the control unit/touch screen with sharp-edged or hot objects. This causes damage to the display.
- The operation of the devices with power sources, products, etc., which are subject to hazardous materials regulations, or could have a negative impact on the health of the operating personnel when using equipment modified by the user, are not permitted.

Note for decommissioning

When the firing device is not in operation, move the lift into the firing chamber and switch off the device at the main switch. Closing the firing chamber protects the insulation and prevents moisture absorption.

Note for cleaning

- Before each cleaning operation, remove the power supply plug!
- In all cleaning work, you must never use any detergents or flammable liquids.
- Do not operate the control unit with sharp-edged or hot objects. This causes damage to the touch screen.

Note for repairs

Repairs and servicing of any kind may only be carried out by specially trained personnel.

Pictogram		
Hazardous voltage	This pictogram warns the user about hazardous voltage. Before opening the unit, always disconnect the device from the mains current by unplugging the AC adapter.	
Hot surface	This pictogram warns the user about hot surfaces that can cause burns.	
Separate disposal	Dispose of electrical and electronic equipment separately, not with household waste. The black bar under the "wheeled bin" symbol indicates that the device was placed on the market after August 13, 2005. Please note that the device is subject to European Community Directive 2002/96/EC (WEEE) and the national laws valid in your country, and it must be disposed of accordingly. Contact your dealer if you need to dispose of the device.	
Note	This pictogram warns of hazardous situations with the risk of personal injury or damage to the device.	

105269/E_V01

The operating manual can be found on the VITA website: www.vita-zahnfabrik.com

VITA