# VITA SMART.FIRE

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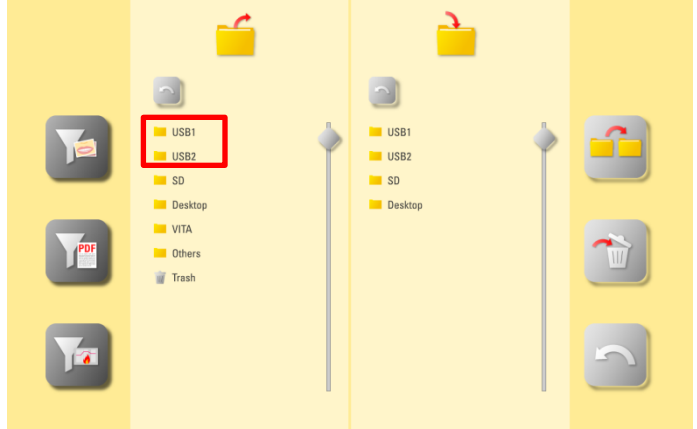
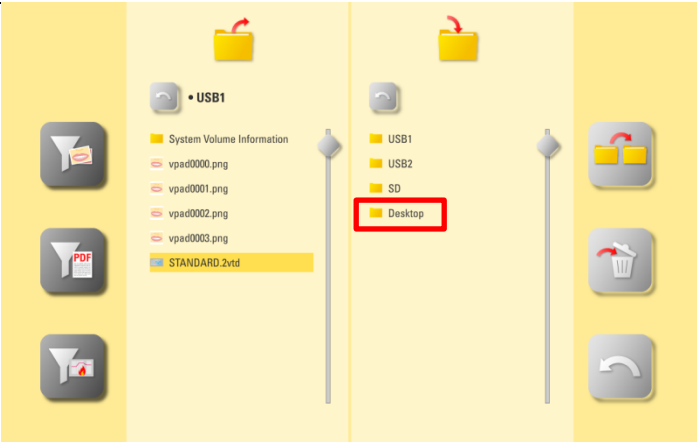
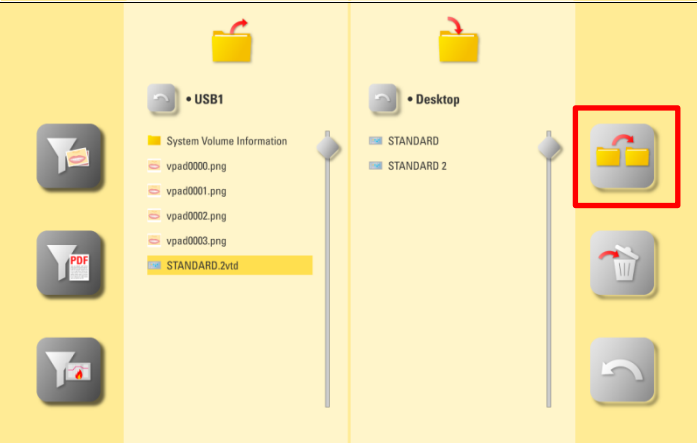
Paso	Introducción por el usuario	Función del aparato
4.1	Pulse el botón  .	 <p>The screenshot shows the vPad control panel interface. At the top, there is a temperature display showing 167°C and a home button. Below that, there are several icons for different functions: a 500°C icon, a home button, a folder icon, and a question mark icon. The question mark icon is circled in red. Below these are several icons for different materials and processes, including VITABLOCS and SUPRINITY PC. At the bottom right, there is a 'STANDARD' button.</p>
4.2	Pulse el botón  .	 <p>The screenshot shows the vPad control panel interface. At the top, there is a temperature display showing 167°C and a home button. Below that, there are several icons for different functions: a 500°C icon, a home button, a folder icon, and a question mark icon. The question mark icon is circled in red. Below these are several icons for different materials and processes, including VITABLOCS and SUPRINITY PC. At the bottom right, there is a 'STANDARD' button.</p>

4.3	<p>Compruebe mediante la versión de software si se ha instalado correctamente el nuevo software.</p> <p>En caso de que no se indique la versión actual, será preciso repetir la actualización.</p>	 <p>The screenshot shows a software interface with the following data:</p> <ul style="list-style-type: none"> <li><b>Device information:</b> <ul style="list-style-type: none"> <li>Dental furnace software version: 01.05.1008</li> <li>Dental furnace serial no.: 2920170145</li> <li>vPad serial no.: 2720170148</li> <li>vPad software version: 180327-full</li> <li>vPad free space: 2.2GB</li> </ul> </li> <li><b>Operating data:</b> <ul style="list-style-type: none"> <li>Operating hours: 82</li> <li>Heating hours: 11</li> <li>Heating hours of firing muffle: 7</li> <li>Number of finished firing programs: 49</li> <li>Aging of firing muffle (%): 0</li> </ul> </li> <li><b>Export:</b> <ul style="list-style-type: none"> <li>Processdata</li> <li>Operating data</li> <li>Device information</li> <li>All data</li> </ul> </li> </ul>
-----	--	---

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5.3	<p>Seleccionar el puerto USB.</p>	 <p>The screenshot shows a file manager window with a sidebar on the left containing icons for Home, Documents, Desktop, and Applications. The main pane displays a list of drives: USB1, USB2, SD, Desktop, VITA, Others, and Trash. USB1 and USB2 are highlighted with a red rectangular box.</p>
5.4	<p>Seleccionar el escritorio exportado.</p>	 <p>The screenshot shows the file manager window with the USB1 drive selected in the sidebar. The main pane displays the contents of the drive, including 'System Volume Information', several 'vpad0000.png' files, and a folder named 'STANDARD.2vtd'. The 'STANDARD.2vtd' folder is highlighted with a yellow selection bar.</p>
5.5	<p>Seleccionar la carpeta "Escritorio".</p>	 <p>The screenshot shows the file manager window with the USB1 drive selected in the sidebar. The main pane displays the contents of the drive, including 'System Volume Information', several 'vpad0000.png' files, and a folder named 'STANDARD.2vtd'. The 'STANDARD.2vtd' folder is highlighted with a yellow selection bar. In the sidebar, the 'Desktop' folder is highlighted with a red rectangular box.</p>
5.6	<p>Pulsar el botón  para importar el escritorio.</p>	 <p>The screenshot shows the file manager window with the Desktop drive selected in the sidebar. The main pane displays the contents of the drive, including 'STANDARD' and 'STANDARD 2' folders. The 'STANDARD' folder is highlighted with a yellow selection bar. In the sidebar, the Import icon (two overlapping yellow rectangles) is highlighted with a red rectangular box.</p>

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