VITA SUPRINITY® PC

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VITA SUPRINITY® PC

Crystallizing and glazing in the CEREC SpeedFire

- VITA SUPRINITY PC has been approved for crystallization in the CEREC SpeedFire from software version CEREC 4.4.4 or higher and CEREC Premium software version 4.4.4.

Available programs are:

- Crystallizing
- Crystallizing / Glazing

The maximum number of 3 crowns must not be exceeded.

The maximum firing chamber size must be observed.

The enclosed glaze support (Dentsply Sirona) must be used for crystallizing.

- Only the VITA AKZENT Plus GLAZE LT SPRAY is approved for glazing VITA SUPRINITY PC in the CEREC SpeedFire.

The following program is available:

- Glazing

No more than 3 glaze firings can be carried out.

The enclosed glaze support (Dentsply Sirona) must be used for glazing.

- The program is automatically selected by the CEREC software when crystallizing / glazing in the CEREC SpeedFire.
- Only the enclosed CEREC SpeedFire firing supports "glaze support single unit or glaze support multi unit," and may be used for crystallization, combination firing (crystallizing / glazing) and glaze firing.
- A small quantity of CEREC SpeedPaste is used for fixing/positioning the restoration on the firing support.
Note: The less CEREC SpeedPaste is used, the better the result. However, to avoid direct contact of the firing support with the restoration, a small quantity of CEREC SpeedPaste must be used for fixation.
- Then the restoration on the firing support is placed onto the upper door insulation (central position).
- To ensure correct use of the CEREC SpeedFire, please read and follow the instructions of the manufacturer (Sirona Dental Systems).

VITA