


VITA vPad with 10.1" display + furnace

(to 04/2017)

To operate the

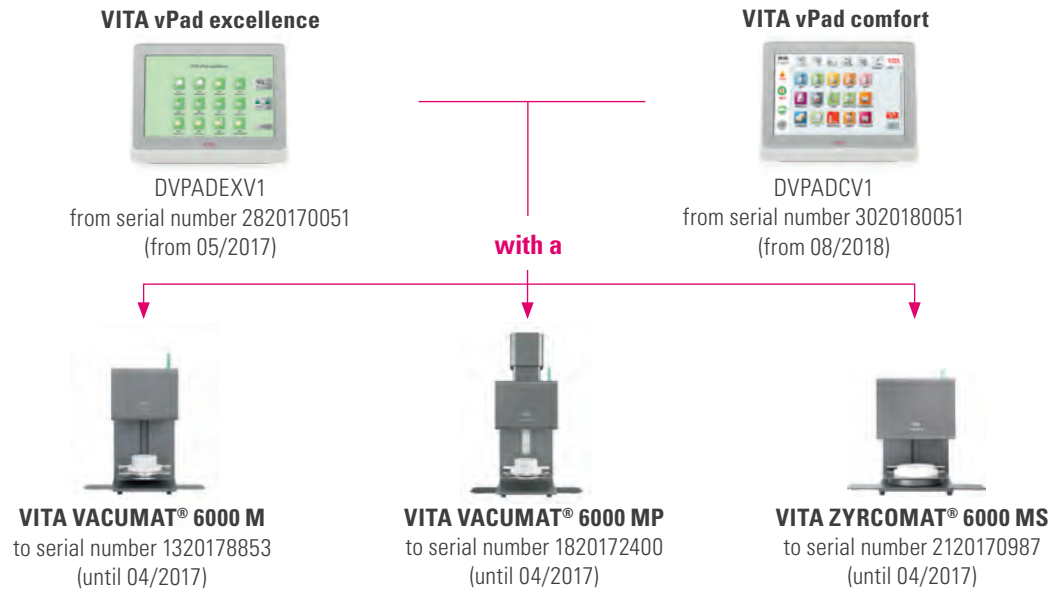

Single Operation

In single operation (= **WITHOUT VITA SWITCHBOX**) a power injector cable is absolutely required.

To start, proceed as follows:



1. Injector
2. Connecting cable (vPad excellence - Injector)
3. Connecting cable (Injector - furnace)
4. Adapter



Injector/connecting cable
(yellow point + ring)



New vPad/connecting cable
(blue point + ring)



1/5 = on-switch / off-switch only for
the vPad, furnace is switched
on/off via the main switch




Use upper port
(until 04/2017)

VITA vPad excellence with 10.1" display + VITA SWITCHBOX + furnaces

(for all years)

To operate the

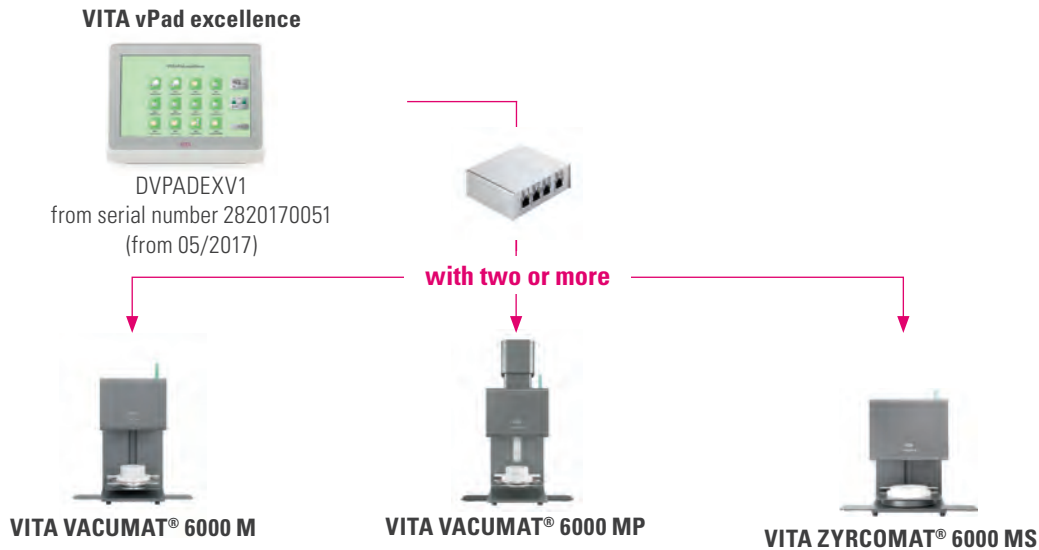

Multiple furnance control

With two or more furnaces of models VITA VACUMAT 6000 M, VITA VACUMAT 6000 MP and VITA ZYRCOMAT 6000 MS via a **VITA SWITCHBOX** (prod. no. D47009 / multiple furnace control) an injector cable (prod. no. D63100) is required.

To start, proceed as follows:



1. Injector
2. Connecting cable (vPad excellence - Injector)
3. Connecting cable (Injector - SWITCHBOX)
4. Connecting cable (SWITCHBOX - furnace)
5. Adapter



Injector/connecting cable
 (yellow point + ring)



New vPad/connecting cable
 (blue point + ring)



1/6 = on-switch / off-switch only for
 the vPad, furnace is switched
 on/off via the main switch



Use upper port
 (until 04/2017)



Use upper port
 (from 05/2017)

VITA