

# VITA vPad with 10.1" display + furnace

(from 05/2017)

## To operate the

### VITA vPad excellence



DVPAD EXV1  
from serial number 2820170051  
(from 05/2017)

### VITA vPad comfort



DVPAD CV1  
from serial number 3020180051  
(from 08/2018)

with a



**VITA VACUMAT® 6000 M**  
from serial number 1320178854  
(from 05/2017)



**VITA VACUMAT® 6000 MP**  
from serial number 1820172401  
(from 05/2017)



**VITA ZYRCOMAT® 6000 MS**  
from serial number 2120170988  
(from 05/2017)

## Single operation

In single operation ( = **WITHOUT VITA SWITCHBOX** ) no power injector cable is required.

The vPad is connected directly to the furnace.


To start, proceed as follows:



The vPad is connected directly to the furnace



New vPad/connecting cable  
(blue point + ring)

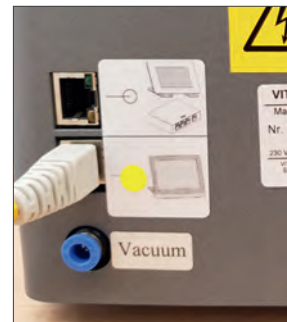

Furnace/connecting cable  
(yellow point + ring)




Use lower port  
(from 05/2017)

# VITA vPad with 10.1" display + furnace

(to 04/2017)

## To operate the

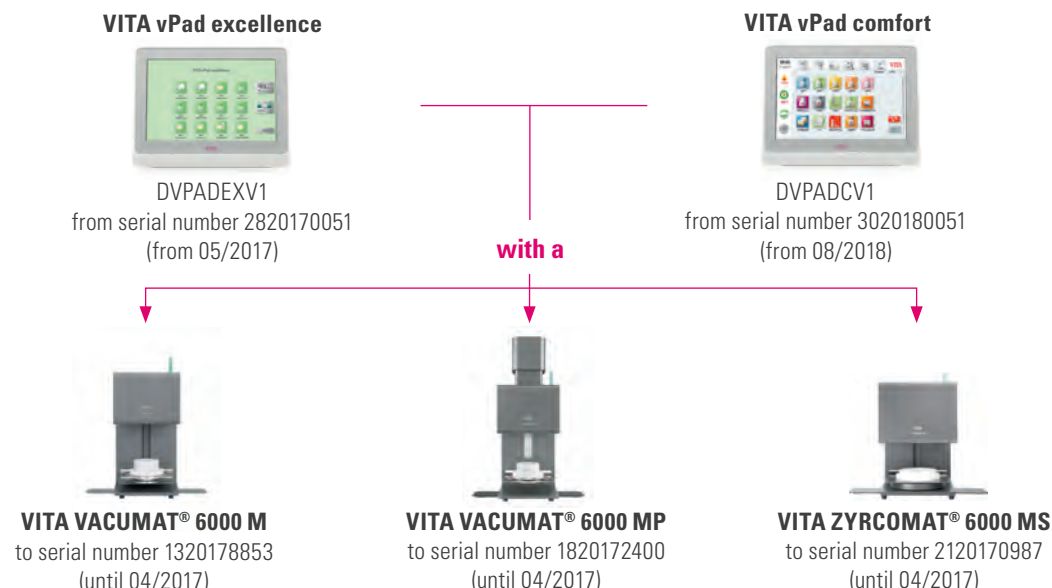

## Single Operation

In single operation ( = **WITHOUT VITA SWITCHBOX** ) a power injector cable is absolutely required.

To start, proceed as follows:



1. Injector
2. Connecting cable (vPad excellence - Injector)
3. Connecting cable (Injector - furnace)
4. Adapter



Injector/connecting cable  
(yellow point + ring)



New vPad/connecting cable  
(blue point + ring)



1/5 = on-switch / off-switch only for  
the vPad, furnace is switched  
on/off via the main switch



Use upper port  
(until 04/2017)

**VITA**