

VITA vPad with 10.1" display + furnace

(from 05/2017)

To operate the

VITA vPad excellence



DVPAD EXV1
from serial number 2820170051
(from 05/2017)

VITA vPad comfort



DVPAD CV1
from serial number 3020180051
(from 08/2018)

with a



VITA VACUMAT® 6000 M
from serial number 1320178854
(from 05/2017)



VITA VACUMAT® 6000 MP
from serial number 1820172401
(from 05/2017)



VITA ZYRCOMAT® 6000 MS
from serial number 2120170988
(from 05/2017)

Single operation

In single operation (= **WITHOUT VITA SWITCHBOX**) no power injector cable is required.

The vPad is connected directly to the furnace.


To start, proceed as follows:



The vPad is connected directly to the furnace



New vPad/connecting cable
(blue point + ring)

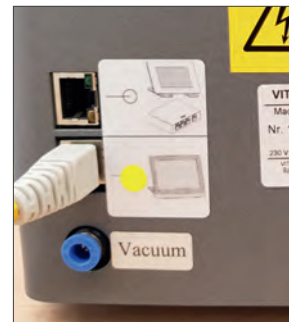

Furnace/connecting cable
(yellow point + ring)




Use lower port
(from 05/2017)

VITA vPad excellence with 10.1" display + VITA SWITCHBOX + furnaces

(for all years)

To operate the

VITA vPad excellence

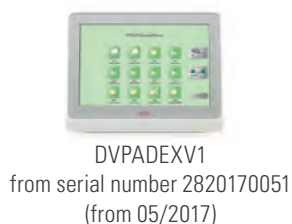

with two or more



VITA VACUMAT® 6000 M



VITA VACUMAT® 6000 MP



VITA ZYRCOMAT® 6000 MS


Multiple furnace control

With two or more furnaces of models VITA VACUMAT 6000 M, VITA VACUMAT 6000 MP and VITA ZYRCOMAT 6000 MS via a **VITA SWITCHBOX** (prod. no. D47009 / multiple furnace control) an injector cable (prod. no. D63100) is required.

To start, proceed as follows:



1. Injector
2. Connecting cable (vPad excellence - Injector)
3. Connecting cable (Injector - SWITCHBOX)
4. Connecting cable (SWITCHBOX - furnace)
5. Adapter

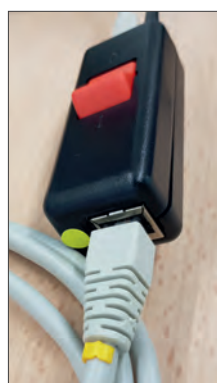

Injector/connecting cable
(yellow point + ring)



New vPad/connecting cable
(blue point + ring)



1/6 = on-switch / off-switch only for
the vPad, furnace is switched
on/off via the main switch



Use upper port
(until 04/2017)




Use upper port
(from 05/2017)

VITA vPad with 10.1" display + furnace

(to 04/2017)

To operate the

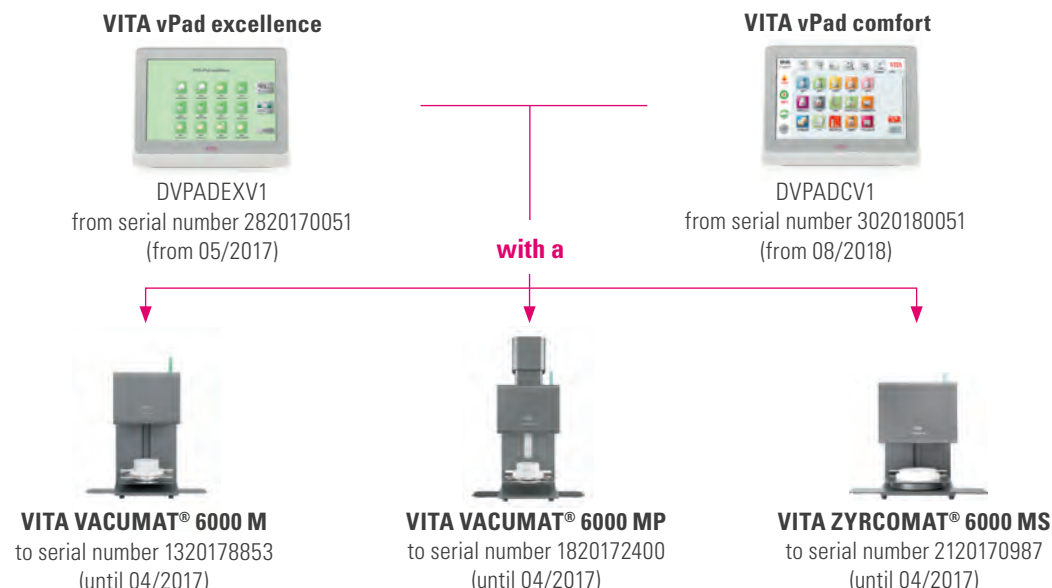

Single Operation

In single operation (= **WITHOUT VITA SWITCHBOX**) a power injector cable is absolutely required.

To start, proceed as follows:



1. Injector
2. Connecting cable (vPad excellence - Injector)
3. Connecting cable (Injector - furnace)
4. Adapter



Injector/connecting cable
(yellow point + ring)



New vPad/connecting cable
(blue point + ring)



1/5 = on-switch / off-switch only for
the vPad, furnace is switched
on/off via the main switch



Use upper port
(until 04/2017)

VITA