


# VITA vPad excellence with 10.1" display + VITA SWITCHBOX + furnaces

(for all years)

## To operate the

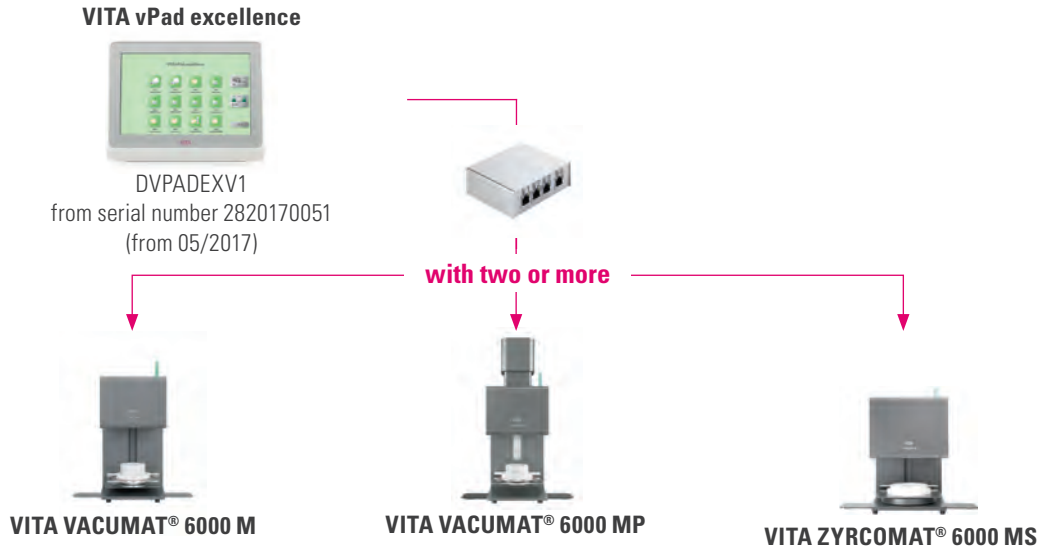


## Multiple furnance control

With two or more furnaces of models VITA VACUMAT 6000 M, VITA VACUMAT 6000 MP and VITA ZYRCOMAT 6000 MS via a **VITA SWITCHBOX** (prod. no. D47009 / multiple furnace control) an injector cable (prod. no. D63100) is required.

To start, proceed as follows:



1. Injector
2. Connecting cable (vPad excellence - Injector)
3. Connecting cable (Injector - SWITCHBOX)
4. Connecting cable (SWITCHBOX - furnace)
5. Adapter

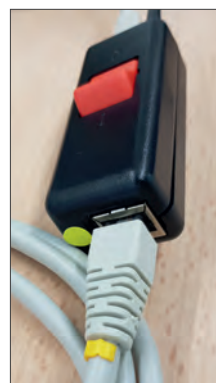

Injector/connecting cable  
 (yellow point + ring)



New vPad/connecting cable  
 (blue point + ring)



1/6 = on-switch / off-switch only for  
 the vPad, furnace is switched  
 on/off via the main switch



Use upper port  
 (until 04/2017)



Use upper port  
 (from 05/2017)

**VITA**