VITA AMBRIA®

VITA AMBRIA® INVEST P/F

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Instructions for use

Mode d'emploi

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Istruzioni d'uso

VITA – perfect match.

VITA

**Symbolerklärung / Symbol explanation / Description des symboles /
Explicación de símbolos / Spiegazione dei simboli**

Nur für professionelle Anwender /
For professional use only /
Réservé à un usage professionnel /
Solo para usuarios profesionales /
Solo per uso professionale

Hinweis auf Gebrauchsanweisung /
See instructions for use /
Voir mode d'emploi /
Referencia a instrucciones de uso /
V. istruzioni d'uso

Temperaturrempfindlich /
Temperature-sensitive /
Thermosensible /
Sensible a la temperatura /
Sensibile alle temperature

Hersteller /
Manufacturer /
Fabricant /
Fabricante /
Produttore

Herstelldatum /
Manufacturing date /
Date de fabrication /
Fecha de fabricación /
Data di produzione

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1 USE

- VITA AMBRIA INVEST has been specially developed for pressing VITA AMBRIA press pellets in a high-speed process.
- Due to the special composition, a reaction layer on the ceramic surface is almost completely avoided.
- Any remaining reaction layer can be easily blasted and the restorations are protected during divesting.
- The time-consuming treatment of the restorations after blasting with acid is no longer necessary.

2 TECHNICAL DATA

Mixing ratio (powder: liquid)	100 g : 23 ml
Mixing time (under vacuum)	60 s
Max. processing time span	5 - 9 min
Setting time (Vicat)	5 - 12 min
Total expansion (linear) = setting expansion + thermal expansion	0.3 – 1.2%
Compressive strength (depending on the concentration of the mixing liquid)	4 – 15 MPa

3 STORAGE

- Store the investment material cool and dry at between 3 to max. 25°C, ideally at 20°C.
- Protect the liquid from freezing.
- Under the recommended conditions, the shelf life of the investment material and the liquid is 36 months each.

Protect against freezing! 3°C

Please note:

- Do not use investment materials from a cold or warm place immediately (storage rooms, silos, storage containers or similar)!
- Fill up the stock in time – before the last carton/pack is used up – so that the mass has reached the recommended room temperature when used.
- Do not use newly delivered packs immediately, but allow them to reach the recommended temperature first.
- Regular control of the temperature in the „investment“ working area.
- Make sure to keep an ample stock (especially the liquid)

so that deliveries are not necessary during the winter months (danger of frost!).

4 DANGER AND SAFETY INFORMATION

IMPORTANT!

Contains quartz and cristobalite

When processing the VITA INVEST P powder:

- Work with object extraction and use respiratory protection if ventilation is inadequate.
- Do not inhale dust.
- This material contains quartz, which may cause lung damage or silicosis with prolonged or repeated exposure if inhaled.
- May cause mechanical irritation of the skin and mucosa of the eye and respiratory tract.
- Do not mix the powder with other materials.
- Dispose of the contents/container in accordance with local regulations.

When processing the VITA INVEST F liquid:

- Important! Mixing liquid is alkaline!
- If mixing liquid comes into contact with skin, rinse immediately with copious amounts of water!
- If mixing liquid comes into contact with the eyes, rinse with copious amounts of water while keeping the eyelid open. In case of concerns: consult an ophthalmologist.
- Do not mix the liquid with other liquids.
- Protect the liquid from freezing!

When heating over 200°C:

- Formation of ammonia vapors
- In the case of local contact, irritation and possibly caustic effect on the skin and mucosa (eyes, respiratory tract, and gastrointestinal tract after swallowing) can be expected.

5 INVESTING

5.1 Preparation

- Check the mixing bowl and mixing spatula for cleanliness and the agitator for operability.
- If wax relaxants are used, no liquid residues must adhere to the wax object.
- Investment ring and base must also be clean.
- Check the position of the wax object in the investment ring for height, position, sprue, etc.

5.2 Measuring the VITA INVEST F liquid mixing ratios

Investment ring size	100 g investment ring	200 g investment ring
Powder	100 g	200 g
Liquid	23 ml	46 ml

Recommended concentrations of the mixing liquid when using wax

Indication	VITA INVEST F liquid [%]	dist. water [%]
Crowns	60 - 70	40 - 30
Three-unit bridge	65 - 75	35 - 25
Veneer / table top	50 - 60	50 - 40
Inlay (one-/two-surface)	35 - 45	65 - 55
MOD inlay	40 - 50	60 - 50

Note:

- Numerous processing parameters, such as temperature, modelling material, vacuum mixer, etc., can affect the fit.
- This may lead to deviations from the above recommendations.

Mixing ratios for different concentrations of the mixing liquid

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5.3 Mixing

- First pour precisely dosed liquid into the bowl.
- Add the appropriate contents to the bag.
- Carefully pre-mix manually for 20 to 30 seconds.
- Close mixing bowl and mix mechanically under vacuum for a further 60 seconds.

Note:

- If any investment material or liquid is spilled, the procedure should be repeated.
- VITA AMBRIA INVEST P must not come into contact with gypsum or gypsum-containing materials, as these can have a negative effect on the properties of the investment material.

Tip:

- For the mixing of VITA AMBRIA INVEST, use separate mixing bowls and spatulas.
- Old mixing bowls can extract 2 - 3 ml of liquid when dry, so always rinse out the bowl with water first and then wipe it dry gently!

5.4 Investing

- Allow mixed investment material to flow into the investment ring under slight vibration, avoiding air inclusions.
- After the wax model has been covered with investment material, stop the vibration immediately.

Note:

- The thin-bodied consistency and the processing time of 5 - 9 minutes allow effortless investment of even difficult restorations without time pressure.

6 CURING / SETTING TIMES

- The investment ring should be placed after 20 minutes, at the earliest, but no later than 30 minutes, after the start of mixing at the final temperature of 850°C.

Note:

- A curing time of more than 30 minutes can adversely affect the fit and lead to cracking.

Tip:

- Set the timer for at least 20 minutes in order to be able to adhere to the minimum required time period.

7 PREHEATING / HEATING

Heating times when wax is used

Investment ring size	100 g	200 g
Setting time	20 - 30 min	20 - 30 min
Placement temperature	850°C end temperature	850°C end temperature
Holding time at end temperature	50 min	75 min

Tip:

- If three 100 g or two 200 g investment rings or more are in the preheating furnace, the holding time must be increased by 15 minutes.

Heating when using acrylics

Investment ring size	100 g	200 g
Setting time	20 - 30 min	20 - 30 min
Placement temperature	Max. 250°C	Max. 250°C
Holding time at 250°C	60 min	60 min
Heating to end temperature	10°C / min	10°C / min
End temperature	850°C	850°C
Holding time at end temperature	50 min	75 min

8 PRESSING

- Please pay close attention to the VITA AMBRIA press instructions in connection with the instructions for use of your press furnace.

9 DIVESTING

- After pressing, allow the investment ring to cool to room temperature.
- The investment ring must not be quenched in cold water.
- Never divest ceramic objects with the divesting pliers.
- Rough divestment with Al_2O_3 (50 μm) at 4 bar pressure.
- Then fine-divest and remove the reaction layer with aluminum oxide blasting agent (50 μm) or glass blasting beads at max. 2 bar pressure.
- No further processing or treatment with acid is required.

Note:

- After heating, the investment material loses its dust-reducing properties.
- Wear a mouth guard when divesting.

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H. Rauter GmbH & Co. KG