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DE

### VITA ADIVA® F-CEM Dualhärtendes Befestigungskomposit

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### VITA ADIVA® F-CEM

Dual-curing adhesive composite

VITA ADIVA F-CEM a **dual-curing**, low viscosity, very radiopaque microhybrid composite with a high filler content for the permanent fully adhesive bonding of restorations made of ceramic, hybrid ceramic, composite and metal to hard tooth substance. VITA ADIVA F-CEM fulfils the requirements of **DIN EN ISO 4049**.

#### Composition

VITA ADIVA F-CEM is based on Bis-GMA-based resins, catalysts, stabilizers, pigments, and inorganic filler particles in a distribution of 0.05 - 1 µm. The filler content is 61% by weight or 41% by volume.

#### Indication

VITA ADIVA F-CEM is suitable for the permanent and fully adhesive bonding (in other words, including phosphoric acid etching and bonding system with VITA ADIVA T-BOND) of inlays, onlays, veneers, crowns, and bridges made of ceramic, hybrid ceramic, and composite to hard tooth substance.

#### Contraindication

For proven allergies to components of VITA ADIVA F-CEM.

#### Side effects

Systemic side effects are not known.

#### Preparation of the VITA ADIVA F-CEM syringe

Remove the cap on the double syringe (**dispose of it, do not re-use it!**) and replace it with the enclosed 1:1 mixing tip. Secure the mixing tip by turning it 90° to the side. The material can be applied directly from the mixing tip.

#### Note:

After storing the material in the refrigerator, it should only be used after it has reached room temperature.

#### The processing time (at 23°C / 73°F) in self-curing mode is 2:00 minutes from the beginning of mixing.

#### Note:

The used mixing tip serves as the cap until the next application. The first 2-3 mm of material that comes out of the mixing tip (around the quantity of a peppercorn) should be thrown away. This applies for each new mixing.

### 1. Bonding of inlays, onlays, veneers, and crowns made of hybrid ceramic (VITA ENAMIC), feldspar ceramic (e.g. VITABLOCS), and glass ceramic (e.g. VITA SUPRINITY PC)

#### Preparation of the restoration

##### 1.1 Etching with hydrofluoric acid gel

Please observe the instructions for use of VITA ADIVA CERA-ETCH, 5% hydrofluoric acid gel.

##### 1.2 Application of silane bonding agent

Apply suitable silane bonding agent (e.g. VITA ADIVA C-PRIME) on the etched surfaces of the restorations.

Observe the instructions for use of VITA ADIVA C-PRIME silane bonding agent.

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A methacrylate-based adhesive composite (e.g. VITA ADIVA F-CEM) can be applied to a surface prepared in this way, according to the instructions for use.

### 2. Bonding of restorations made of zirconia (e.g. VITA YZ)

#### Preparation of the restoration

##### 2.1 Sandblasting with Al<sub>2</sub>O<sub>3</sub>

Sandblast the zirconia surfaces to be bonded or veneered with high-grade aluminum oxide (Al<sub>2</sub>O<sub>3</sub>, max. 50 µm, blasting pressure 2 bar). Next, clean the sandblasted surface.

##### 2.2 Application of zirconia bonding agent

Place VITA ADIVA ZR-PRIME zirconia bonding agent on a mixing palette, apply an even layer with a disposable brush or microbrush on a surface that has been prepared according to the manufacturer's instructions, and let the bonding agent work for **10 s**. Slightly disperse the volatile components with compressed air.

A methacrylate-based adhesive composite (e.g. VITA ADIVA F-CEM) can be applied on a surface that has been prepared this way, according to the instructions for use.

### 3. Bonding of metal restorations

#### Preparation of the restoration

##### 3.1 Sandblasting with Al<sub>2</sub>O<sub>3</sub>

Sandblast the restoration to be bonded or veneered with high-grade aluminum oxide (Al<sub>2</sub>O<sub>3</sub>, 50 µm, blasting pressure 2 bar). Next, clean the sandblasted surface in an ultrasonic bath and dry with oil-free air.

##### 3.2 Application of metal bonding agent

Place VITA ADIVA M-PRIME metal bonding agent on a mixing pallet. Apply an even layer with a disposable brush or microbrush to the preconditioned bonding surfaces and let it work for **10 s**. Slightly disperse the volatile components with compressed air.

A methacrylate-based adhesive composite (e.g. VITA ADIVA F-CEM) can be applied on a surface that has been prepared in this way, according to the instructions for use.

### 4. Bonding of composite restorations

#### Preparation of the restoration

Please observe the corresponding processing instructions of the manufacturers of the composite materials.

#### Pre-treatment of the tooth substance

##### Keep the treatment area dry.

The use of a rubber dam is recommended.

Remove the temporary and clean the tooth substance with pumice powder. Try in of the restoration.

##### Etching the tooth substance with phosphoric acid etching gel

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