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## Instructions for use

### VITA VIONIC® WAX milled synthetic wax blank

Please read the instructions for use before using the product.

#### Intended use

VITA VIONIC WAX is a blank made of millable and dimensionally stable synthetic wax with a high melting temperature for the fabrication of full-sized wax try-ins and wax-ups.

#### Patient target group

No restrictions

#### Indication

Indication range:

- VITA VIONIC WAX White is used for the fabrication of full-sized try-ins. They are used to test the mid- line, occlusion plane and phonetics.
- VITA VIONIC WAX Pink is used for the fabrication of denture bases for try-in. For this purpose, denture teeth are fixed in the milled cavities. All parameters can be checked, and if necessary, corrections can be implemented.

#### Contraindication

- Not suitable for the fabrication of definitive dental restorations.

#### Product application/control

##### Workflow:

VITA VIONIC WAX White:

1. Clamp the wax disc,
2. Mill the wax disc (maxilla and mandible),
3. Separate the try-in,
4. Check the try-in against the master model,
5. Intraoral try-in,
6. Implement any required changes digitally using the CAD software,
7. If necessary, perform another try-in.

VITA VIONIC WAX Pink:

1. Clamp the wax disc,
2. Mill the wax disc (maxilla and mandible),
3. Separate the try-in,
4. Use wax to fix denture teeth (for conventional denture teeth, prior CAM modification is required),
5. Check against the master model,
6. Intraoral try-in,
7. Scan in any required changes as a new regis- tration, and implement digitally using the CAD software,
8. Alternative conventional method of finalizing the dental restoration.

The product must be processed at room temperature.

##### System requirements:

For the VITA VIONIC WAX system requirements, see www.vita-zahnfabrik.com/vionicwax

Detailed information on processing can be found in the document "Instructions for Use I Full Version" at www.vita-zahnfabrik.com/vionicwax

#### Recommended product combinations

For the try-in with VITA VIONIC WAX Pink, the pre- fabricated digital VITA VIONIC VIGO denture teeth or the millable VITA VIONIC FRAME denture teeth can be used.

## Storage/disposal

Store at room temperature. Store in a dry location. Protect from direct sunlight and temperatures above 25°C.

The countersigned products labelled with a picto- gram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper, plastics) must be disposed of using appropriate recycling systems.

## Technical data

Physical properties	Unit of measure	Value
Solidification point	°C	83
Drop point	°C	89
Flash point	°C	>140
Penetration at 25 °C	0.1 mm	9
Viscosity at 120 °C	mPas	9.7 - 10.1

Chemical composition	Wt%
Paraffin waxes	65 - 85
Pentaerythrityl	10 - 30
Polyethylene	5 - 12

The technical data are averages and may fluctuate slightly between delivered batches.

## General risks/reporting incidents

There are no known side effects to date.

Information on the general risks associated with dental treatments and on reporting serious incidents in connection with medical devices can be found at www.vita-zahnfabrik.com/service\_risks

The safety data sheets can be downloaded at www.vita-zahnfabrik.com/vionicwax or requested by fax at (+49) 7761-562-233.

## Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunc- tion with materials or equipment from other manu- facturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not a required component of the product. Date of issue of this information: 04/20 After the publication of this information for use any previous versions become obsolete. The current version can be found at www.vita-zahnfabrik.com VITA Zahnfabrik has been certified and the following products bear the mark:

VITA VIONIC® WAX