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DE

VITAFOL® H Silikon-Isoliermaterial

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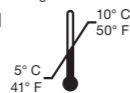
EN

VITAFOL® H silicone insulation material

Please read the instructions for use before using the product.

Intended use

VITAFOL H contains silicone insulation material for the protection of acrylic and ceramic teeth during denture fabrication and can be used on wax models prior to embedding.

Patient target group

No restrictions

Indication

N/A (not a medical product).

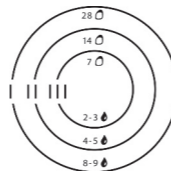
Contraindication

N/A (not a medical product)

Product application/control

Please adhere to the following workflow when working with this product:

- Apply the paste and the hardener liquid to the dosing block and mix for approx. 1 minute using a spatula

- Apply the mixture to individual areas that require insulation or spread it over the entire wax model and allow it to flow into the interdental spaces
- While the insulation film is still damp, sprinkle the bonding crystals over the surface evenly
- Leave it to harden at room temperature for approx. 8-12 minutes. Then invest the model with plaster in the flask as usual
- Once the plaster has set, separate the flasks and boil out the wax
- During the divestment process, teeth and areas that have been treated with VITAFOL H come out of the cuvette without any plaster on them

Detailed information on working with this product can be found in the document "Instructions for Use I Full Version" at www.vita-zahnfabrik.com/vitafolh

Recommended product combinations

VITAFOL H consists of VITAFOL H paste, VITAFOL H hardener and VITAFOL H bonding crystals, and should only be used in this combination.

Storage/disposal

VITAFOL H paste and VITAFOL H hardener must be stored at a refrigerator temperature of between 5°C (41°F) and 10°C (50°F).

The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems.

Safety information

- After removal, immediately seal the tube and bottle properly.

- If the hardener liquid thickens up, this is due to a failure to observe correct product usage.
- The hardener liquid may become slightly cloudy or mottled; this will not affect its performance.
- The volume of the hardener bottle matches the volume of the paste tube.
- The tube is made of flexible plastic and allows easy kneading of the paste after long periods without use.

General risks/reporting incidents

Information on the general risks associated with dental treatments and on reporting serious incidents in connection with medical devices can be found at www.vita-zahnfabrik.com/service_risks

Hazardous substances

VITAFOL H hardener

IMPORTANT!

- | | |
|-----------|---------------------------------------------------------------------------------------------------|
| H226 | Flammable liquid and vapor. |
| H319 | Causes serious eye irritation. |
| H332 | Harmful if inhaled. |
| H335 | May cause respiratory irritation. |
| H361d | May cause harm to unborn children in pregnant women. |
| H372 | Prolonged or repeated exposure will cause damage to organs. |
| P210 | Keep away from heat, hot surfaces, sparks, open flames and other sources of ignition. No smoking. |
| P260 | Do not breathe in dust/fume/gas/mist/vapors/spray. |
| P280 | Wear protective gloves/protective clothing/eye and face protection. |
| P337+P313 | In case of persistent eye irritation, seek medical advice or consult a doctor. |

Safety data sheets can be downloaded at www.vita-zahnfabrik.com/vitafolh or requested by fax from (+49) 7761-562-233.

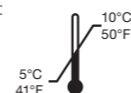
Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials or equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not a required component of the product. Date of issue of this information: 04/20

After the publication of this information for use, any previous versions become obsolete. The current version can be found under www.vita-zahnfabrik.com

VITA Zahnfabrik has been certified and the following products bear the mark:

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