**VITA CAD-Waxx**

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**VITA CAD-Waxx**

Please read the instructions for use before using the product.

**Intended use**

VITA CAD-Waxx is an acrylate polymer that burns out without leaving any residue. It is used for the computer-aided fabrication of wax-ups for pressing or casting, the fabrication of drilling templates for implant restorations and the CAD/CAM fabrication of provisional models to verify the fit.

**Patient target group**

No restrictions.

**Indication**

- Casting:
  - VMK crown and bridge substructures
  - Full-cast crowns and bridges
- Pressing

**Contraindication**

VITA CAD-Waxx is not suitable for the fabrication of temporaries and must not be fixed in the mouth on either a temporary or a permanent basis.

**Product application/control**

- Please observe manufacturer's instructions issued by the respective system providers during the CAD/CAM processing of VITA CAD-Waxx.
- For the fabrication of molded parts made of VITA CAD-Waxx for pressing/casting, observe the minimum wall thicknesses required by the respective material manufacturers.
- Please use a cross-cut carbide bur to cut molded parts out of the blank and to remove posts or cones.
- For details on the further processing of VITA CAD-Waxx molded parts for the fabrication of cast and pressed objects, follow the respective manufacturer's instructions.

**Storage/disposal**

No special storage conditions.

The products labeled with a pictogram for hazardous substances must be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems.

**Technical data**

Chemical composition: 100 % PMMA

**General risks/reporting incidents**

Information on the general risks associated with dental treatments and on reporting serious incidents in connection with medical devices can be found at [www.vita-zahnfabrik.com/service\\_risks](http://www.vita-zahnfabrik.com/service_risks)

Safety data sheets can be downloaded at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com) or requested by fax from (+49) 7761-562-233.

**Disclaimer**

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability for damage incurred as a result of using the product in conjunction with materials or equipment provided by other manufacturers that is not compatible or not authorized for use with our product. The VITA Modulbox is not a required component of the product. Date of issue of this information: 04/20

After the publication of this information for use, any previous versions become obsolete. The current version can be found under [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com)

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